

DRSK 

DRSK EXTRACTION™



Premium products
at
non-premium prices

Summary

DRSK Extraction™ is a tooth model which can be used for practicing tooth extraction. Practicing on this model gives the students a tactile feel of resistance to extracting force close to one experienced in a real-life exodontia procedure. DRSK Extraction allows use of elevators to perform luxation as the first step in exodontia and therefore makes possible a complete simulation of this procedure.

Using DRSK Extraction in a preclinical setup enables the students to adequately learn the principal extraction maneuvers and build up the manual skill of properly applying pressure to the target tooth.

DRSK Extraction tooth is designed to be installed in DRSK Jaw, which is available as a separate product. Practicing extraction on this model without placing it inside DRSK Jaw is not recommended.

DRSK Extraction is available in the form of following teeth:

- Maxillary central incisor
- Maxillary first premolar
- Maxillary first molar
- Mandibular central incisor
- Mandibular canine
- Mandibular first premolar
- Mandibular first molar

DRSK Extraction is manufactured in Sweden.

Features

Remarkable features of DRSK Extraction

- DRSK Extraction is a cost-effective tool to teach exodontia.
- DRSK Extraction is anatomically-shaped in both root and crown.
- DRSK Extraction is available in the form of various teeth.

Possible applications for DRSK Extraction:

- Training undergraduate students
- Evaluating undergraduate students

Ordering

- Minimum order quantity is 20 pieces.
- Each order can contain one type or a combination of different types of models.
- To place an order, send an email to sales@drsk.com.
- To view the detailed list of DRSK products, please check the following links:
 - DRSK RCT: <https://www.drsk.com/rct.pdf>
 - DRSK Restorative: <https://www.drsk.com/res.pdf>
 - DRSK Extraction: <https://www.drsk.com/ext.pdf>
 - DRSK Jaw: <https://www.drsk.com/jaw.pdf>

Instructions for use

In order to perform extraction on DRSK Extraction, it needs to be installed in DRSK Jaw (For more information on this product, refer to the [DRSK Jaw catalogue](#)). After each extraction, the remaining part of DRSK Extraction can be removed from DRSK Jaw and discarded.

To place DRSK Extraction into DRSK Jaw:

- Select the DRSK Extraction model you want to work on.
- Push it through the corresponding opening at the base of DRSK Jaw until it sits snugly in place.
- Drive the pin which comes with DRSK Jaw through the small hole on the side of the jaw, next to where the tooth model is now placed. The pin has to be pushed all the way in until only a few millimeters of it stick out. Now DRSK Extraction model must be secure inside DRSK Jaw.

To remove DRSK Extraction from DRSK Jaw:

- Detach the jaw model from the phantom head (if it is already attached).
- Pull the pin out of the jaw model. This releases the remaining part of DRSK Extraction. Put the pin aside for future use.
- Press down on remaining part of DRSK Extraction model to take it completely off the jaw.

Note: DRSK Extraction models have a limited shelf-life of 180 days (after their production date). To ensure the best educational experience, it is recommended to practice on DRSK Extraction within the shelf-life indicated on its package. Practicing on DRSK Extraction which are past their shelf life is expected to be of limited educational value. Also they should be kept in dark places.



Safety precautions

When working on DRSK models, always observe these points:

- Throughout the practice you **MUST**:
 - Wear eye protection.
 - Put on a mask.
 - Have gloves on.





- Practicing on DRSK models has to be done in places where safety standards are maintained.
- People with a history of allergy (particularly, allergy to resins) need to consult with their physicians and follow their suggested precautions.
- Keep the models out of reach of children and pets.
- Models are recommended to be used only for the purposes indicated in DRSK catalogues.
- Tools and equipment have to be used on DRSK models with the same precautions required when working in a clinical set-up.
- Always use clean and non-infected tools on DRSK models.
- DRSK models and their accessories have to be disposed of in a proper way in accordance with local rules and regulations.


The material used in DRSK models is proved as non-cancerogenic.




Extraction model


<i>Maxillary central incisor</i>	Article Code	Color	Crown
 <p>Number of roots: 1</p>	EXT-21-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		


<i>Maxillary first premolar</i>	Article Code	Color	Crown
 <p>Number of roots: 2</p>	EXT-24-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		


<i>Maxillary first molar</i>	Article Code	Color	Crown
 <p>Number of roots: 3</p>	EXT-26-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		

Extraction model



<i>Mandibular central incisor</i>	Article Code	Color	Crown
 <p>Number of roots: 1</p>	EXT-31-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		

<i>Mandibular first premolar</i>	Article Code	Color	Crown
 <p>Number of roots: 1</p>	EXT-34-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		

<i>Mandibular first molar</i>	Article Code	Color	Crown
 <p>Number of roots: 2</p>	EXT-36-01	White	Yes
	<p>Suggested for: University courses, Demonstrating extraction instruments</p>		

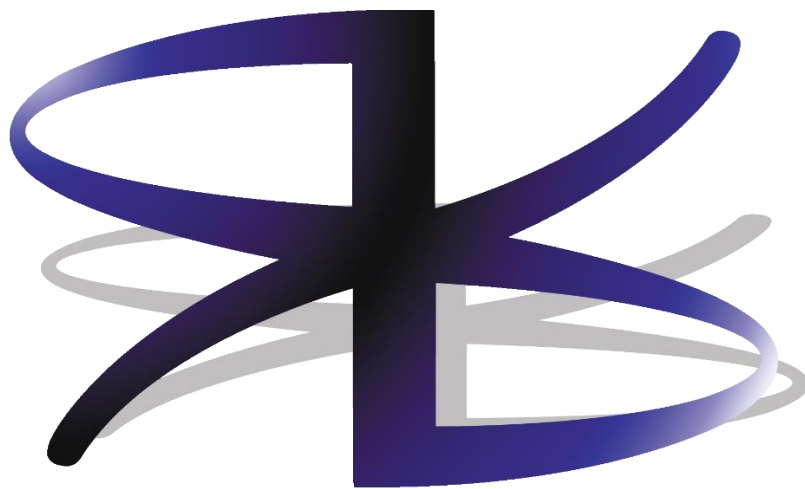

Dental and Digital Innovative Solutions

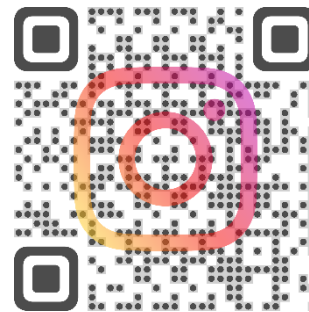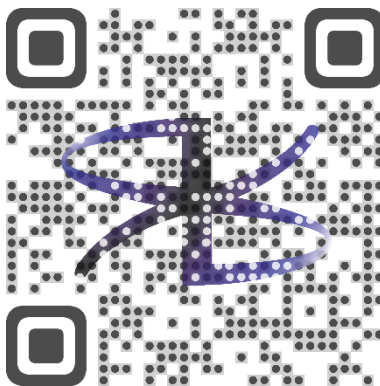

DRSK Group AB
Enhörningsv. 6C,
281 43 Hässleholm,
Sweden
<https://www.drsk.com>

Terms of use:

*Any content contained in this info package including the text and images may not be copied or reproduced in whole or in part in any manner or on any media to any person without the prior written consent of DRSK Group AB.
Copyright © 2019-2023 by DRSK Group AB, all rights reserved. The DRSK logo is a trademark of DRSK Group AB.*